AKITO

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Massage table user manual

Akito massage tables are a guarantee of the highest quality and comfort of work. Their correct use and maintenance ensure durability for many years.

Additional information

PL 1 - 6

CE conformity mark

Medical device

Read the user manual

Manufacturer/Producer

Authorized representative of European Community

EN 7 - 11

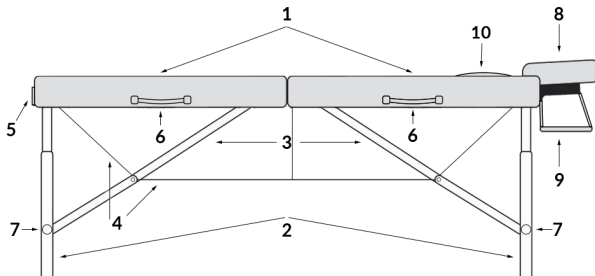
1. Purpose

EN

Akito folding rehabilitation tables are designed in accordance with the instructions for use and the generally applicable occupational safety and health and safety regulations, for professional treatment carried out by a doctor, physiotherapist, massage therapist or other trained personnel. They can be used both in medical and doctor's surgeries, as well as in the home environment. Akito tables enable professional users to correctly, safely and effectively carry out treatments rehabilitation, physiotherapy, physical therapy, massage in order to treat or alleviate the course of illnesses and effects of injuries. The construction of massage tables allows therapist easy and full access to the patient during treatments.

2. Massage table's construction

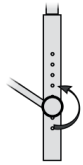
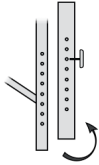
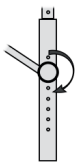
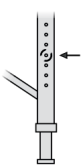
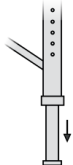
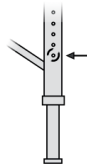
1. Upholstered table top
2. Table legs
3. Table leg spacers
4. Cable tension (cable system)
5. Case lock
6. Transport handles
7. Height adjustment system
8. Headrest
9. Armrest
10. Face hole plug (optional on selected models)

3. Unfolding the massage table

1. The table in its folded state constitutes a suitcase.
2. Open the lock (5).
3. Fold back the table tops (1).
4. Remove the equipment (8, 9) from the inside of the table.
5. Fold the legs (2) outwards as far as possible.
6. Place the table on a firm and level surface (before loading, the center of the table top should be tilted approximately 3 cm upwards).
7. Check the tension of the cable system (4).
8. Set the correct height of the table according to the table "Height adjustment".
9. Fit the headrest (8) and the other accessories (9).
10. The lifting of the moving part of the table is done by unlocking the moving part (optional on selected models).

Tab.1 Height adjustment

1	2	3	Table name	Description of operations
			Hera Hekate	1. Unscrew the nut. 2. Remove the mobile leg from the retaining bolts, move the mobile leg into the desired position. 3. Screw on the nut.
			Apollo Hermes Light Herkules Zeus	1. Push in the locking pin. 2. Move the mobile leg into the desired position. 3. Squeeze the pin out.

4. Patient positioning

The therapist should determine the correct position on the table for the patient, assist the patient and help the patient to get off the table safely at the end of the treatment.

5. Assembling the table

1. Remove the head rest (8) and the other accessories (9).
2. Move the table from the working position to the folding position (lateral position on rubber feet). Fold the table legs paying attention to the correct position of the cables and their fastening parts.
3. Place the equipment.

6. Maintenance, cleaning and disinfection of structures

1. Painted Parts:

Clean with products that do not contain abrasives.

2. Wood and plywood:

- Surfaces made of these materials should be cleaned with a soft, slightly damp cloth
- Do not use any chemical cleaners.

Please note: As a natural product, wood may have: differences in grain, colour tones, natural inclusions such as streaks, dots, etc., shades of colour. This does not detract from the natural appearance of the wood. This does not lower the quality of the product, but rather emphasizes its natural origin. Such features do not constitute basis for complaint.

3. Upholstery:

- Avoid contact with oils. Use oil-resistant covers or waterproof sheets.
- Use and store away from direct sources of fire and heat.
- Do not use products containing solvents, pastes, waxes, sprays, strong detergents, natural and organic leather cleaners.
- Do not expose to prolonged sunlight.
- If the upholstery becomes soiled with oils or creams, immediately clean the surface with mild detergent, e.g. grey soap and a soft cloth or sponge and wipe dry.

- In the case of heavier greasiness, use a 25% solution of ethanol and wipe gently with a soaked gauze pad. Finally wipe the cleaned area with a damp cloth and wipe dry. If necessary, disinfect with preparations dedicated to disinfection of PVC, PU padding. After the upholstery has been disinfected, it should be thoroughly dried before being placed in the upholstery cover. Otherwise permanent staining of upholstery can occur.

WARNING: Before using any product other than a mild detergent, test the effect in an invisible area and carry out the cleaning very carefully.

7. Operational notes

1. The table should be stored in a clean, dry, enclosed area.
2. The table is designed to be used by one person only at the same time.
3. It is forbidden to:
 - move the table with the patient,
 - sit down abruptly or jump on the table,
 - sit down, kneel down and lean on the headrest, shelves and armrests.
4. When folding the table, pay attention to the alignment of the cables, do not snap them.
5. Keep and use away from direct sources of fire and heat.
6. Proper use ensures long and trouble-free operation.

8. Technical specifications

Tab.2 Technical specifications

Table	Length	Width	Height adjustable by leaps	Dimensions after folded	Weight without equipment	Dynamic strength*	Static strength**
Hera	184 cm	60 cm	58 - 79 cm	92x60x20 cm	13,0 kg	204 kg	1000 kg
Apollo	184 cm	60 cm 71 cm	60 - 81 cm 60 - 81 cm	92x60x20 cm 92x71x20 cm	12,8 kg 13,5 kg	204 kg	1000 kg
Hermes light	184 cm	60 cm	61 - 81 cm	92x60x20 cm	10,3 kg	204 kg	1000 kg
Hekate	184 cm	64 cm	60 - 86 cm	92x64x18 cm	13,5 kg	204 kg	1000 kg
Herkules	184 cm	71 cm	60 - 80 cm	92x71x18 cm	12,0 kg	204 kg	1000 kg
Zeus	184 cm	60 cm 71 cm	61 - 81 cm	92x60x18 cm 92x71x18 cm	13,5 kg 14,0 kg	204 kg	1000 kg

*Dynamic strength - measured with a uniformly distributed dynamic load, simulating the stresses occurring during a classic massage

**Static strength - measured under uniformly distributed stationary load

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2. COVERAGE AND PERIOD OF THE WARRANTY

- 1. The Seller guarantees the good quality and well-functioning of the equipment for which the warranty card has been issued, for the period stated in this warranty.
- 2. The warranty repair does not include the actions described in the user manual, which the Purchaser is obliged to carry out himself.
- 3. The Purchaser loses his warranty entitlements in case of:
 - a) the Seller verifies that, during the warranty period, the equipment was repaired outside the Authorised Service Centre,
 - b) the Purchaser makes any structural modifications to the equipment without the Seller's written consent
 - c) use of the equipment after a design fault has been discovered
- 4. Regardless of the loss of warranty entitlements in the cases described in section 3, the Seller shall not be liable for any damage to property or person that may be caused by the use of the equipment.
- 5. The warranty does not exclude, limit or suspend any rights arising from the non-conformity of the goods with the contract.
- 6. The coverage of warranty protection is limited to the territory of Poland.
- 7. Defects or damage to the structure of the equipment resulting from causes depended on the equipment, discovered during the warranty period, will be repaired free of charge within 14 days from the date of delivery of the equipment to the Seller or an Authorised Service Centre.

3. THE WARRANTY DOES NOT INCLUDE:

- a) damages resulting from improper or inconsistent use, storage, maintenance or transport of the equipment;
- b) colour differences between exposed and unexposed parts to the sunlight;
- c) effects of chemical or abrasive agents, atmospheric pollution or accidental punctures;
- d) presence of stains caused by greasy or coloured substances (grease, creams, lotions, oils);
- e) permanent staining with colouring agents, such as permanent inks, markers or other non-permanent dyes used in clothing production (e.g. blue jeans)
- f) mechanical damages

4. TERMS AND CINDITIONS OF WARRANTY

1. If the Purchaser wants to use the warranty entitlements, all three conditions must be met:

- a) presentation of the invoice or other proof of purchase of the equipment by the Purchaser,
- b) report the defect within 14 days of its discovery,
- c) return the goods in their original packaging at the Seller's expense via Seller's representative or a courier company servicing the Seller. The Purchaser should agree with the Seller the date and method of collecting the goods. Shipping costs not agreed with the Seller shall be borne by the Purchaser.

2. The warranty period shall be extended by the duration of the repair, calculated from the date of delivery of the equipment to the Seller or Authorised Service Centre, until the date on which the equipment is available to collect by the Purchaser.

3. The guarantee is limited to repairing the product or, in the second place, replacing the product. The Seller does not accept any liability for indirect loss or damage.

4. The Seller is not liable for damage to the equipment caused during the transport service by the transport company. It is the Purchaser's responsibility to check the technical condition of the equipment upon receipt.

5. If the transportation of the equipment was commissioned by the Seller and the equipment is damaged, the Purchaser should:

- a) notify the Seller within 3 days of receipt of the shipment of any damage to the equipment which has occurred during transport, in order to enable the Seller to make a claim to the transport company,
- b) provide the Seller with a complaint report written down between the buyer and the transport company.

6. The warranty rules contained in this document are the only and original rules of the Falcon product warranty.

7. The guarantee period for all Falcon products is 24 months, excluding upholstered items, for which the guarantee period is 12 months.

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PRODUCT NAME:

.....

DATE OF PURCHASE AND THE SELLER'S STAMP:

.....

THE SHIPPING ADDRESS FOR THE NEW OR REPAIRED PRODUCT,
INCLUDING A TELEPHONE NUMBER:

.....

.....

THE DATE OF REPAIR:

.....

DESCRIPTION OF THE REPAIR:

.....

.....

.....

AKITO

Shanghai Sintang Industrial Co., LTD
Linsheng Road 136, Tinglin Town,
Jinshan District,
Shanghai 201505, China
Tel: +86-21-3721 0909